

IpsiHand[®] Candidate Identification Guidelines

FDA INDICATION FOR USE

Must meet all of the below requirements

- Age 18+
- ≥ 6 months post-stroke
- Hemiparesis or hemiplegia in upper limb

CONTRAINDICATIONS

If any of the following applies, the patient is not an appropriate candidate

- Severe spasticity in elbow, wrist, or hand (Modified Ashworth Scale ≥ 3)
- Rigid contractures in wrist/digits
- Skull defects from craniectomy/craniotomy

Clinical Considerations


Proceed if all apply or clinical judgment supports use

- Has sufficient PROM in wrist/digits for device fit
- Can follow simple instructions (written/visual)
- Has adequate vision to interact with tablet
- Can maintain upright head position for 60 min (certain types of head support may be accommodated)
- Skin on arm/hand is intact and tolerant to wear handpiece
- EEG sensors must make contact with the scalp and some thick hairstyles may make this difficult.

Home Setup Requirements

- Must have stable Wi-Fi
- Caregiver or support available, if needed, for setup
- Ability to use tablet with English instructions

Note: Severe neglect, confusion, or memory deficits may limit device use.

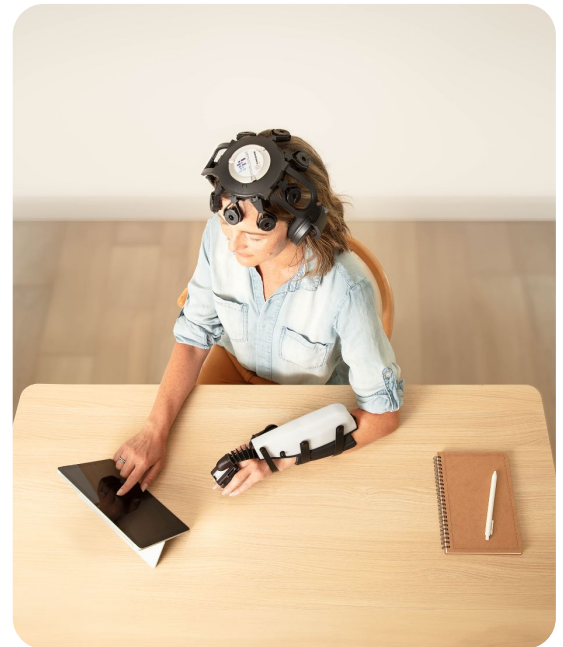

Questions About Clinical Candidacy?

If you're unsure if your patient is a good clinical fit, let us know which specific criteria require clarification. We can arrange a remote clinical assessment. Contact us at support@neurolutions.com or 833-813-4774.

IpsiHand is manufactured by Neurolutions, Inc., a Kandu, Inc. company.